**OPED UK Ltd**

Unit 5  
Avro Business Centre  
Avro Way  
Bowerhill  
Melksham  
SN12 6TPQ  
United Kingdom  
Phone +44 (0)1380 722177  
Fax +44 (0)1380 710095  
enquiries@oped.biz  
www.oped-uk.com

**OPED GmbH**

Medizinpark 1  
83626 Valley/Oberlindern  
Germany  
Fon +49 (0)80 24/60 81 8-210  
Fax +49 (0)80 24/60 81 8-299  
mail@oped.de  
www.oped.de

**OPED AG**

Hinterbergstrasse 26  
6312 Steinhausen  
Switzerland  
Fon +41 (0)41/748 53 35  
contact@oped.ch  
www.oped.ch

**Orthotic & Prosthetic  
Centre Pty. Ltd.**

OPC Health an OPED company  
26-32 Clayton Rd  
Clayton VIC 3168  
Australia  
Phone +61 (0)1300 672 937  
sales@opchealth.com.au  
www.opchealth.com.au

**OPED Medical, Inc.**

444 Tom White Road  
Braselton, GA 30517  
USA  
Phone +1 (770) 945-0150  
Fax +1 (770) 945-0370  
info@opedmedical.com  
www.opedmedical.com

attraction | 02/2023

**VADOplex**<sup>®</sup> by OPED  
VASCULAR IMPULSE TECHNOLOGY

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**Foot Pad**

Single patient – multiple use.  
Non-Sterile, 100% Latex-Free

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Place foot as shown by graphic in the pad. Note Left and Right side. The inflation chamber must be directly under the arch of the foot.

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- ⚠ Check regularly and in short intervals the patients comfort and skin irritations, particularly for patients with poor circulation, fragile skin, insensitive extremities, Diabetes mellitus and patients under anticoagulation therapy. Investigate tenderness.
- ⚠ If necessary reduce applied pressure settings, apply additional padding or discontinue the therapy.
- ⚠ Check regularly to make sure the impulse is felt directly under the arch of the foot.
- ⚠ Do not walk or weightbear on the Pad while inflated.
- ⚠ Do not elevate the limb during treatment.
- ⚠ Only inflate Pad when fitted.
- ⚠ Keep away from fire sources.

**Information about indications and contraindications can be found in the instructions for use of VADOPlex.**

In the event of complaints relating to the product, such as damage or problems with the fit, please contact the manufacturer directly. Only serious events which can lead to a significant deterioration in health or death should be reported to the manufacturer and the responsible authorities of the member state. Serious events are defined in Article 2 (65) of Directive (EU) 2017/745 (MDR).

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Wrap the inside of the sleeve over the top of the foot. Cross the sides of the pad around the bridge of the foot as well and tighten them. Secure with the Velcro strap. The connecting tube must be on the inside of the foot.

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Guide the heel strap around the heel and secure with the Velcro strap at the outside front. Wrap the Velcro strap around the sole of the foot.

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The pads are to be used for one patient only and may neither be used for several patients nor refurbished. Using them for more than one patient or refurbishing them can cause spreading infection chains and wound infections. The function of the pads could be reduced and limit the therapeutic effect.

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