**OPED UK Ltd**

Unit 5
Avro Business Centre
Avro Way
Bowerhill
Melksham
SN12 6TPQ
United Kingdom
Phone +44(0)1380 722177
Fax +44(0)1380 710095
enquiries@oped.biz
www.oped-uk.com

**OPED GmbH**

Medizinpark 1
83626 Valley/Oberlaindern
Germany
Fon +49(0)8024/60818-210
Fax +49(0)8024/60818-299
mail@oped.de
www.oped.de

OPED AG

Hinterbergstrasse 26
6312 Steinhausen
Switzerland
Fon +41(0)41/748 53 35
contact@oped.ch
www.oped.ch

**Orthotic & Prosthetic
Centre Pty. Ltd.**

OPC Health an OPED company
26-32 Clayton Rd
Clayton VIC 3168
Australia
Phone +61 (0)1300 672 937
sales@opchealth.com.au
www.opchealth.com.au

OPED Medical, Inc.

444 Tom White Road
Braselton, GA 30517
USA
Phone +1 (770) 945-0150
Fax +1 (770) 945-0370
info@opedmedical.com
www.opedmedical.com

attraction | 02/2023

VADOplex[®] by OPED
VASCULAR IMPULSE TECHNOLOGY

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Hand Pad

Single patient – multiple use.
Non-Sterile, 100% Latex-Free

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Open the Pad and place the hand on top of the inner side of the Pad, with the thumb located into the cut out provided (see graphic on the Pad bladder). Note Left and Right side.

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- ⚠ Check regularly and in short intervals the patients comfort and skin irritations, particularly for patients with poor circulation, fragile skin, insensitive extremities, Diabetes mellitus and patients under anticoagulation therapy. Investigate tenderness.
- ⚠ If necessary reduce applied pressure settings, apply additional padding or discontinue the therapy.
- ⚠ Check regularly to make sure the impulse is felt directly under the arch of the foot.
- ⚠ Do not walk or weightbear on the Pad while inflated.
- ⚠ Do not elevate the limb during treatment.
- ⚠ Only inflate Pad when fitted.
- ⚠ Keep away from fire sources.

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Information about indications and contraindications can be found in the instructions for use of VADOPlex.

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In the event of complaints relating to the product, such as damage or problems with the fit, please contact the manufacturer directly. Only serious events which can lead to a significant deterioration in health or death should be reported to the manufacturer and the responsible authorities of the member state. Serious events are defined in Article 2 (65) of Directive (EU) 2017/745 (MDR).

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Wrap the end of the cover across the back of the hand, fitting snugly and secure with the velcro closure.

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Wrap the wrist strap around the base of the thumb and secure with the Velcro strap.

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